


Focus: no blurry or rectangular-shaped fixation dots in center of pupil, or blurry photos.

iScreen Home Page


Patient Information

Name/ID....	James Jones
DOB.....	11/9/2014
Dr/Site.....	Iowa City Daycare

Acquisition Complete - Save? (Y / N)



Horizontal Flash



Vertical Flash

Flash: Ready
Images Not Sent: 14

Capacity: 14/2000
Thur Dec 15 2016 11:24:23

Focus Problem:

The white fixation dots are elongated, not a pin-point as they should be.

Facial features can also appear blurry.

Make sure that the two red aiming dots from the iScreen device are coming together to make one red dot on the child's forehead which ensures proper distance from the child.